


Fast Handpiece Maintenance Guide

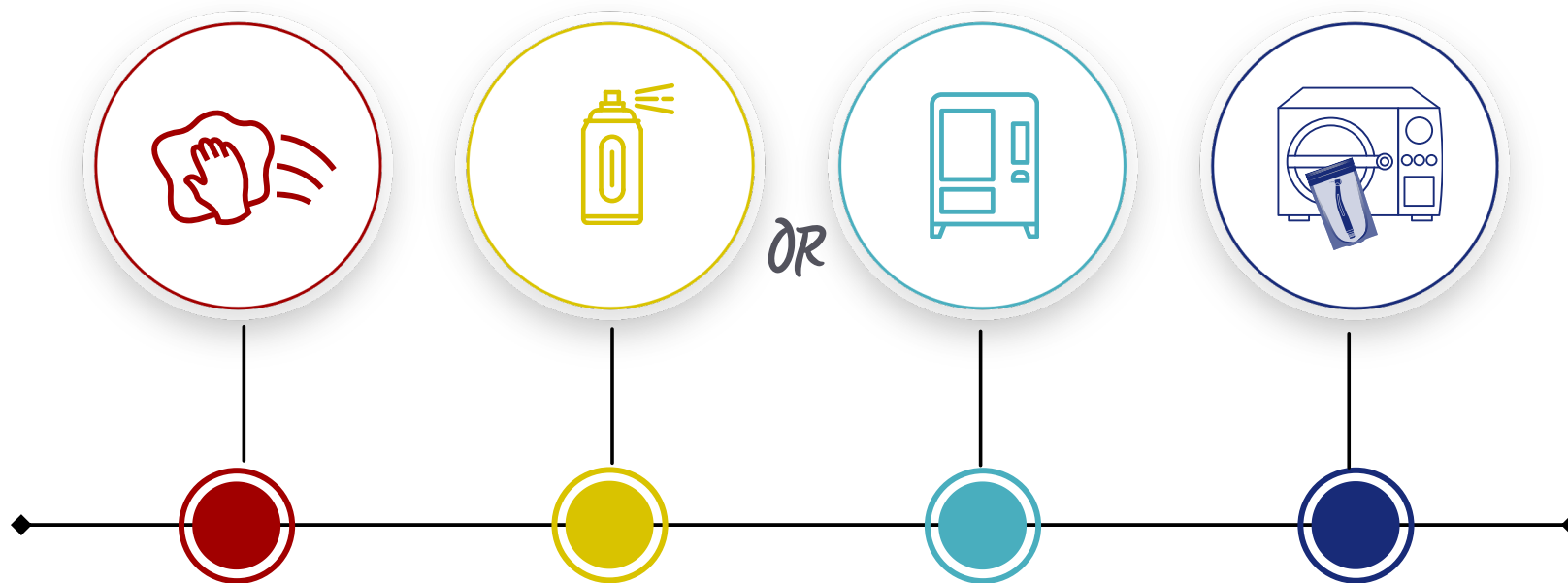

Surface

Wipe down the handpiece with a damp disposable cloth.

- Do not use any chemical solutions for cleaning.
- Do not submerge the handpiece in water, ultrasonic cleaner, or disinfectant.

Spray Lubricant

After selecting the correct adapter for the backend of your handpiece (the front end of the adapter should resemble your dental unit's coupling and click in properly), insert a bur into the handpiece.

Spray lubricant into the handpiece for approximately 2-3 seconds. Count: 1...2...3

Before applying pressure, position the head of the handpiece within a paper towel. If the purged lubricant on the paper towel is not clear, reapply lubricant until the fluid is completely clear.

OR

Automatic Lubrication

After selecting the correct adapter for the backend of your handpiece (the front end of the adapter should resemble your dental unit's coupling and click in properly), insert a bur into the handpiece.

Please follow your automatic oil/maintenance machine's manufacturer instructions.

Bag & Sterilize

Wipe down the exterior of the handpiece with a dry towel to remove any expelled fluid or debris. The handpiece should be completely dry at this point. Bag and sterilize according to the manufacturer's instructions.

- Be sure to allow the handpiece to complete the drying cycle.
- Never sterilize with a bur installed.
- Do not relubricate the handpiece after sterilization.

Recommended Weekly Maintenance:

Remove the bur and lubricate the chuck using the "Spindle" adapter on your can of lubricant, or use a pen droplet oiler.

Use an interproximal brush to clean out the chuck.